SAFETY protection equipment

APA _ ARIA Protection Adaptor OR030100

Compatible masks:

OCEANREEF made masks
(ARIA QR+, ARIA Classic, ARIA Jr, UNO, ARIA 19),
TUSA Sport, all of which are in compliance with
EU Regulation 2016/425.

Ocean Reef Group

MESTEL SAFETY Srl
Via Arvigo, 2
16010 Sant'Olcese
Genova (Italia)
Tel. +39 010 7082011
Fax +39 010 7082099

oceanreefgroup.com
mestelsafety.com

OCEAN REEF Inc
2510 Island View Way
Vista, CA 92081
Phone +1 760 744 9430
Fax +1 760 744 9525
Toll free 1 800 922 1764

WARNINGS: This product can expose you to chemicals including Bisphenol A, which is known to the State of California to cause birth defects or other reproductive harm. For more information, visit www.P65Warnings.ca.gov.

Be environmentally conscious.
Reuse and recycle this packaging.
Dispose properly of the products when throwing away.

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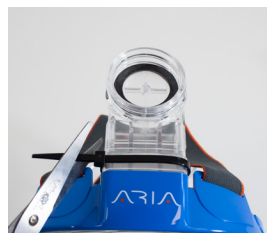
CLEANING & MAINTENANCE

- Remove any filter connected to the adaptor prior to sterilization.
- NEVER use any solvent, cleaner or abrasive product to clean the Mask and the APA Adaptor.
- Clean each product (Mask and APA) by either:

TREATMENT IN DILUTED ETHANOL

1. Place the mask and the APA adaptor in a commercial solution of ethanol (dilution: 3-5% vol/vol) for 5-10 minutes.
2. Possibly, each mask and adaptor should not overlap or cover the other ones.
3. After the sanitification, remove carefully the mask and the APA adaptor from the solution, rinse deeply under fresh water just to remove all the alcohol residuals and allow the perfect drying in a safe and not contaminated environment.

TREATMENT IN DILUTED NaClO

1. Place the mask and the APA adaptor in a commercial solution of NaClO-sodium ipochlorite (dilution: max 2.5-3% vol/vol) for 5-10 minutes.
2. Possibly, each mask and adaptor should not overlap or cover the other ones.
3. After the sanitification, remove carefully the mask and the APA adaptor from the solution, rinse deeply under fresh water just to remove all the ipochlorite residuals and allow the perfect drying in a safe and not contaminated environment.

TREATMENT WITH SANITISING WIPES

1. Clean the mask and the APA adaptor with commercial alcohol based, benzalkonium chloride or ipochlorite based wipes, allowing also a prolonged contact with the different components of the mask.
2. After the sanitification, remove the wipes, disposing them properly, and allow the perfect drying in a safe and not contaminated environment.

TREATMENT IN BOILING WATER

1. Place the mask and the APA adaptor in boiling water for 5-10 minutes.
2. Possibly, each mask and adaptor should not overlap or cover the other ones.
3. After the sanitification, remove carefully the mask and the APA adaptor from hot water, allowing the perfect drying in a safe and not contaminated environment.

TREATMENT IN DISHWASHER

1. Place the mask and the APA adaptor in a dishwasher for one (1) hour at 70° C using dishwasher soap. The dishwasher should be sterilized prior to use. They should be the only objects in the dishwasher.
2. After the sanitification, remove the mask and the APA adaptor from the machine, allowing the perfect drying in a safe and not contaminated environment.

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INSTRUCTIONS

Understand and follow all warnings and instructions.

1. **Prior to use the adaptors, verify that the snorkeling mask properly works and that all parts are present, undamaged, and correctly installed.** NO modifications, alterations or damages must be present on the snorkeling mask. If the mask would show one of the problems listed above, DO NOT proceed with installation of the adaptors since the mask does not work properly.
2. After checking the mask correctly works, before to install the adaptors, clean the mask following the methods reported in the "Cleaning & Maintenance" section of this manual.
3. **The mask and the adaptors CANNOT be used for snorkeling in water.** To use the mask in the water, remove the adaptor and put on the snorkel and the front cover in case in which to be removed.
4. **Never use the mask and the adaptors in an environment poor of oxygen (less than 17% of oxygen present).**
5. NEVER use the mask as personal protection equipment without the APA and the filter (EN148-1 connection) properly installed, or without the RD40 adaptor with the HEPA filter (conical connector 22mm - ISO 5356-1) properly installed as well. **Failure to properly use, store and maintain the mask and the adaptors may result in serious physical injury, illness or death. Mestel Safety does not assume any responsibility about the filters used by customers and about an incorrect use of the parts.**
6. The best way to use the adaptors is put on the mask without any disruption between the face and the seal. Always check the mask fits perfectly before use being in a safe and controlled environment. Wear the mask with the adaptor in the right position over the face and make sure there is no hair in the mask seal areas. The strap must be centered at the top back of the head. Adjust the strap tightening until a secure fit and a perfect seal is reached. Facial hair may prevent a perfect sealing.
7. One size will not fit all faces. For a perfect seal choose your ideal size.
8. Always check that your equipment works properly before the use: try to breath before adjusting the straps.
9. NEVER change, deform or replace any part of the product.
10. To prevent serious injury DO NOT deform or add extensions to the original shape of the adaptors. DO NOT change or remove the valve and the seal between the APA adaptor and the filter connection. DO NOT change or remove the two valves on the oronasal and the one under the cover over the frontal side of the transparent visor (chin of the mask).
11. Never allow open flames or hot air blasts to come into direct contact with the product.
12. In case of discomfort and/or long-time use, if user's breathing becomes uncomfortable, remove the mask with the adaptor in a safe and controlled environment. If at any time the user feels uncomfortable or dizzy, if breathing becomes difficult; if the smell of contaminants enters the mask; if airflow becomes restrictive or completely stops; if the mask becomes foggy or blurry; and/or the user's face, eyes, nose or mouth become(s) irritated, the user must immediately leave the contaminated area and return to a safe location where contamination levels are low and the user can safely remove the mask.
13. The APA adaptor is NOT compatible with any other mask which is not mentioned in this leaflet.
14. The RD40 adaptor is NOT compatible with any breathing system filter other than with an HEPA filter (conical connector 22 mm - ISO5356-1).
15. **Avoid touching the mask and the adaptors if it has not been sanitized to decrease the risk of contact transmission. Pathogens on the surfaces can potentially be transferred by touch to the user's hand, and thus risk infection through subsequent touching of the mucous membranes or the face. Additionally, all parts may be contaminated by other pathogens that are acquired from people that have prolonged environmental survival. These pathogens could then contaminate the hands of the user, and in turn be transmitted to yourself or to others via direct or indirect contact transmission.**
16. **Store the mask with the adaptors in a controlled environment where access to these parts is reduced to prevent damage or contamination of any component. The mask and the adaptors should not be stored in environments where there are potentially dangerous chemicals that may corrode, pollute or permeate them.**

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